



TSI Wheelchair Set-up for Engage APOD™

Standard Set-up for Complete Seating System

The Engage APOD™ can be ordered for use in a standard w/c for patients with normal upper body control.

- High back reclining w/c or Standard w/c; sling seat style chairs only
- Rear anti-tippers
- ELR; *except self propel patients*
- Hemi-height for self propel
- Medium Engage APOD™ cushion; **cushion width = w/c width – NO EXCEPTIONS**
- Secure cushion firmly to w/c by tightening strap around w/c back

Optional items

A. Velcro Closure Thigh Belt for reminding the patient to ask for assistance before attempting to stand. Mount on seat section at 3rd screw from w/c front.

B. Wing-Backs Side Support for the patient who continues to lean despite the Engage APOD™. Mount near top of high-back section below head rest in area of the upper arm.



Adjust ELR on delivery. Feet must be resting flat on the footrest with thigh contacting the front of cushion.

If footrest does not adequately support foot in highest adjustment, patient will require additional foot pad (L-shaped Foot Hugger depth 1" or 3") before using device.



For gaps between shoe and footrest, use L-shaped foot hugger adjustment pad

Engage APOD™ is a registered trademark of The PostureWorks, LLC